

ASEPT Drainage Catheter Dressing Kit

Product Description

The ASEPT Drainage Catheter Dressing is used to protect an indwelling catheter placed in the pleural or peritoneal space. The ASEPT Drainage Catheter Dressing allows the patient to access the drain-line through the plastic slider and zipper seal track without removing the dressing.

Each ASEPT Drainage Catheter Dressing Kit contains the following features:

- ASEPT Drainage Catheter Dressing
- Foam Catheter Pad
- 2 Gloves (Pair)
- 3 Alcohol Pads

Indications for Use

The ASEPT Drainage Catheter Dressing Kit is used to cover and protect drainage catheters and the catheter placement site.

Contraindications

There are no known contraindications.

Warnings

- Stop any bleeding at the site before applying the dressing.
- Do not stretch the dressing during application as tension can cause skin trauma.
- Make sure the skin is dry and free of soap residue and lotion to prevent skin irritation and to ensure good adhesion.
- For single use only. Re-use may result in a non-functional product or contribute to cross contamination.
- Do not use if package is damaged.
- Do not re-sterilize.
- Do not use the dressing on an infected site.

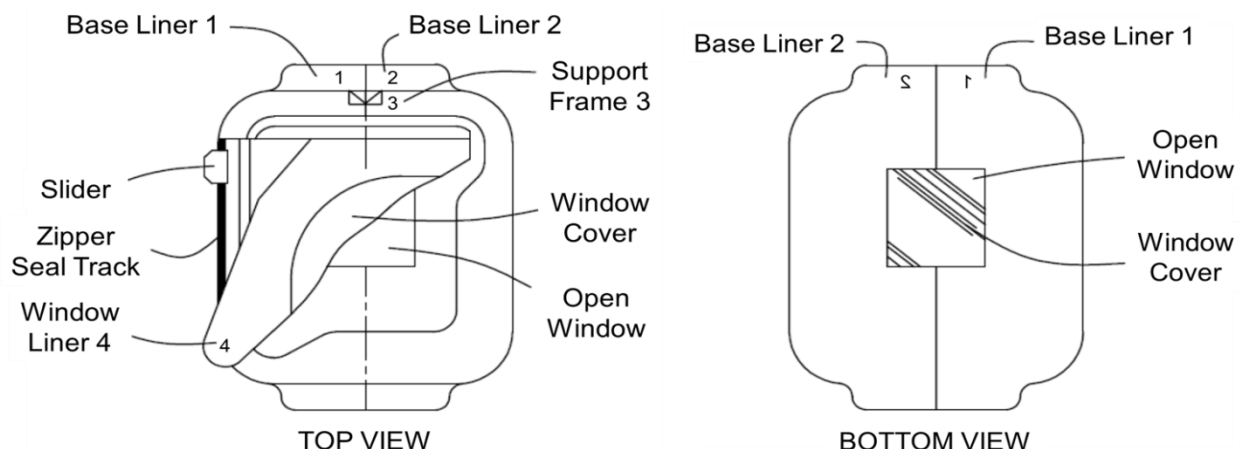
Precautions

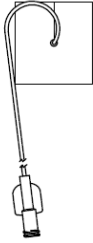
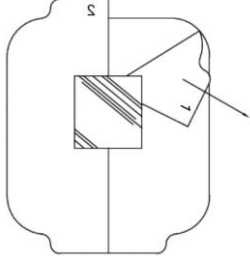
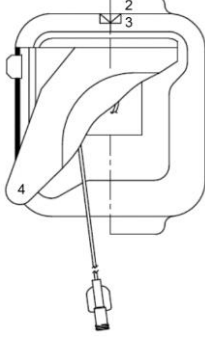
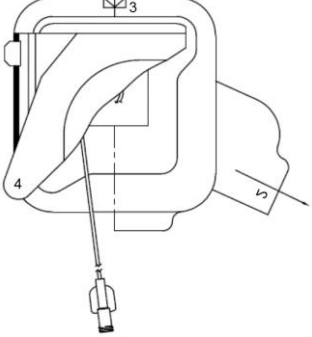
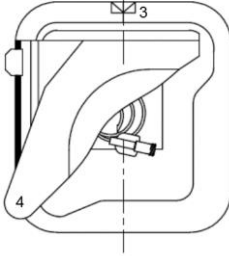
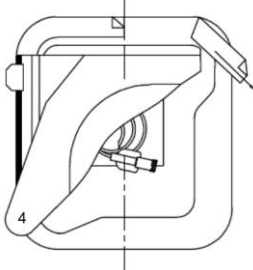
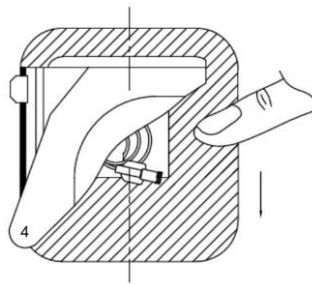
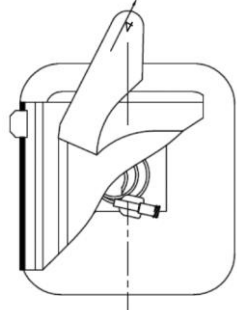
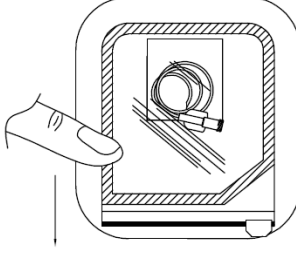
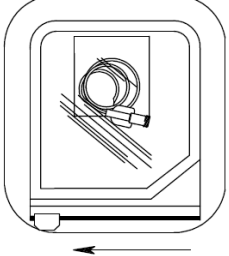
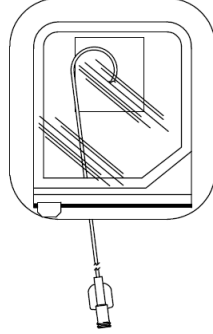
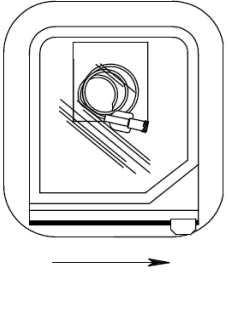
- Site should be observed for signs of infection or other complications.
- If infection is suspected, remove the dressing, inspect the site directly, and determine appropriate medical intervention. Infection may be signaled by fever, pain, redness, swelling, or unusual odor or discharge.
- Change dressing according to institution protocol or when the barrier properties have been compromised. For example, if the dressing is loose or punctured or area under dressing becomes moist, it should be changed.
- This product is not made with natural rubber latex.
- Federal (USA) Law restricts this device to sale by or on the order of a physician.

Application

Site Preparation Recommendations:

- Prepare the site according to institution protocol. If applying the dressing at home, follow instructions below.
 1. Wash your hands with soap and water for at least 1 minute.
 2. Remove the previous dressing and foam catheter pad from over the catheter, if applicable.
 3. Peel open the ASEPT Drainage Catheter Dressing Kit and remove the gloves, alcohol pads, foam catheter pad and dressing.
 4. Pick up one glove by the wrist and put it on. Pick up the other glove by the wrist and put it on the other hand. Both gloves fit on either hand.
 5. Open one of the alcohol pads and clean around the wound site, where the catheter exits the body.
 6. Clean the valve on the end of the catheter.
- Clipping the hair at the site may improve dressing adhesion. Shaving is not recommended.
- Allow all preps and protectants to dry completely before applying dressing to prevent skin irritation and to ensure good adhesion.



<p>Step 1 Place the Foam Catheter Pad at the catheter base.</p> 	<p>Step 2 Remove and discard Base Liner 1 from the dressing.</p> 
<p>Step 3 Center the Open Window over the catheter base and Foam Catheter Pad. Pull the catheter through the Open Window. Apply the dressing over the Foam Catheter Pad.</p> 	<p>Step 4 Remove the remaining Base Liner 2 from the dressing and apply the dressing. Smooth dressing from center to edges to ensure smooth, continuous adhesion to skin.</p> 
<p>Step 5 Coil the catheter and tuck into the corner of the Window Cover. Ensure catheter is not kinked.</p> 	<p>Step 6 Remove the Support Frame Liner 3 from the dressing.</p> 
<p>Step 7 Apply pressure to the base dressing so the dressing adheres to the skin.</p> 	<p>Step 8 Remove the Window Liner 4 from the dressing.</p> 
<p>Step 9 Fold the Window Cover over the catheter. Apply pressure to the perimeter of the window to secure the adhesive to the base dressing. Take care to ensure no gaps in the perimeter seal during application.</p> 	<p>Step 10 To access the catheter, slide the Slider to the "Open" position.</p> 
<p>Step 11 Pull the catheter through the opening in the Zipper Seal Track.</p> 	<p>Step 12 To return, once catheter has been cleaned and closed as per prescribing physician, coil the catheter and place back into the dressing pocket. Ensure catheter is not kinked once inserted. Slide the Slider to the "Closed" position to secure the dressing.</p> 

Removal Instructions

1. Slide Slider to "Open" position and pull the catheter through the opening. Ensure the catheter is fully unwound prior to removal of the dressing.
2. Grasp the dressing base edge opposite the Zipper Seal Track and gently pull it straight out to stretch it and release adhesion. A medical adhesive solvent can also be used to facilitate dressing removal.
3. Care should be taken to not dislodge the catheter or disrupt the skin surface when the dressing is removed. Support the skin and catheter while removing the dressing.

For explanation of the symbols on the labeling and packaging, visit www.pfmmedicalusa.com


VLV Associates, Inc.
34 Troy Road
Whippany, NJ 07981
www.vlvassociates.com

Distributed By:
pfmmedical
1916 Palomar Oaks Way
Suite 150
Carlsbad, CA 92008
(760) 758-8749
www.pfmmedicalusa.com